



# Online Test Menu

## LABORATORY CATALOG

LABCORP  
WASHINGTON/OREGON  
PROVIDENCE AMBULATORY

CODE	TEST NAME	CPT4 CODE
LAB24736	MACROPROLACTIN Container: Gel-barrier tube, red-top tube, lavender-top (EDTA) tube, serum transfer tube, OR green-top (heparin) tube. Temperature: FREEZE Place of Service: Esoterix Endocrinology Laboratory	84146 x2
LAB103	MAGNESIUM Container: Red-top tube, gel-barrier tube, OR green-top (lithium heparin). Do NOT use oxalate, EDTA, or citrate plasma. Temperature: Maintain specimen at room temperature.	83735
LAB2404	MAGNESIUM, RBC Container: Green-top (heparin) tube or lavender-top (EDTA) tube or royal blue-top (EDTA) tube or tan-top lead free tube. Temperature: Room Temperature	83735
LAB406	MAGNESIUM, URINE, 24HR Container: Plastic urine container with 30 mL 6N HCl Temperature: Room Temperature (KBS tablet is acceptable).	83735
LAB2133	MALARIA, NAAT Container: Lavender-top (EDTA) tube or yellow-top (ACD) tube Temperature: Refrigerate	87798
LAB883	MALARIA, SMEAR Container: Glass slide, Lavender-top (EDTA) tube Temperature: Maintain EDTA whole blood specimen at room temperature for no longer than 24 to 48 hours.	87207
LAB1050	MANGANESE Container: Royal blue-top (EDTA) tube Temperature: Maintain specimen at room temperature.	83785
LAB1051	MANGANESE, URINE Container: Plastic urine container, no preservative Temperature: Maintain specimen at room temperature.	82570 83785
LAB1052	MANGANESE, WHOLE BLOOD Container: Royal blue-top (EDTA) tube; submit original tube. Temperature: Maintain specimen at room temperature.	83785
LAB24805	MATERNAL CELL CONTAMINATION, B (NON ORD) Container: Lavender-top (EDTA) tube or yellow-top (ACD) tube Temperature: Room temperature; refrigerated is also acceptable.	81265
LAB13241	MATERNAL SCREEN, 1ST TRIMESTER Container: Serum Gel Tube Temperature: Room temperature	84163 86336 84702
LAB13	MEDICAL CYTOLOGY Container: Sterile container; nongynecologic liquid-based cytology vial Temperature: Refrigerate immediately after collection and ship to laboratory as soon as possible.	88112
LAB24676	MELATONIN, SERUM/PLASMA Container: Red-top tube or lavender-top (EDTA) tube Temperature: Refrigerate	80299
LAB24884	MEPERIDINE (GCMS) REFLEX (REF)	80362
LAB24705	MERCAPTOPYRINE Container: Red-top tube OR green-top (heparin) tube. Gel-barrier tubes are NOT recommended. Temperature: SUBMISSION/TRANSPORT (<3 days): Room Temperature. For storage beyond three days, specimen should be refrigerated or frozen.	80299